


SDR® Plus

Bulk Fill Flowable

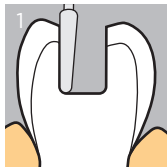


Cavity Preparation

Keep the cavity preparation to a minimum.




Matrix Placement

For Class II restorations place a matrix and wedge. Burnishing of the matrix band will improve contact and contour.

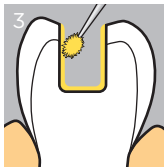

Adhesive Application

Refer to adhesive manufacturer's directions for properly applying adhesive. Light cure taking depth of cavity into account.

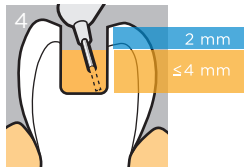

SDR® plus Application


Begin at the deepest portion of the cavity. Gradually withdraw while filling.

SDR® Plus

Bulk Fill Flowable




Caution, consult
instructions for use

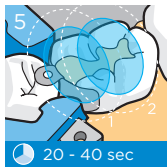

Light Curing


Light cure with a minimum output of 550 mW/cm².

U shade ≥ 20 sec.
A shades ≥ 40 sec.

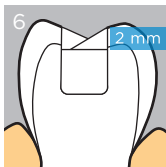

Placement of Universal Composite

Restore occlusal 2 mm with oblique layers to facilitate occlusal contouring. Light cure according to the manufacturer's instructions for use.

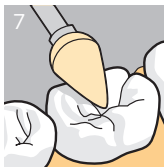

Finishing & Polishing

Contour the restoration using finishing burs or diamonds. Finish the restoration by applying medium pressure and slowly back off to a light

pressure. For a glass-like luster, polish the restoration.

Dentsply DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
www.dentsplysirona.com

© 2017 Dentsply Sirona, Inc.

