

# Neo Spectra™ ST Effects

## Universal Composite Restorative

# Next-Level Esthetics, Simplified

For esthetically demanding anterior cases, Neo Spectra™ ST Effects composite works with Neo Spectra™ ST composite in a simplified layering technique.

- Neo Spectra™ ST Effects opaque **dentin** shades are paired with Neo Spectra™ ST **universal CLOUD shades** when a case calls for added opacity.
- One translucent **enamel** shade creates a highly translucent incisal edge when needed.
- Simplified layering. Made possible by **SphereTEC® technology**.

1 out of 3

people say that the first part of someone's face they notice is their teeth.<sup>2</sup>

OVER  
1/3


of clinicians consider color matching or overall esthetics to be the most challenging aspect of an anterior composite restoration.<sup>3</sup>

D1

One of two Neo Spectra™ ST Effects opaque dentin shades for added opacity.


A2

One of five Neo Spectra™ ST universal CLOUD shades to cover the VITA® range.<sup>1</sup>

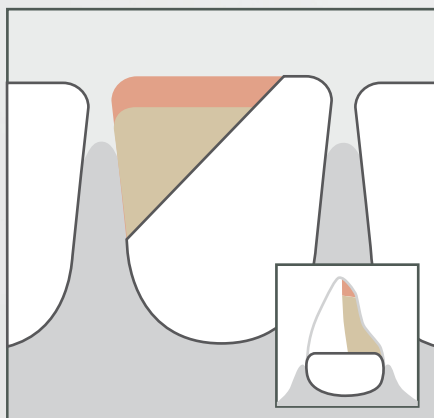


### Opaque Dentin Shade

Restore lingual 1/2 of tooth thickness with opaque dentin and universal CLOUD shade.

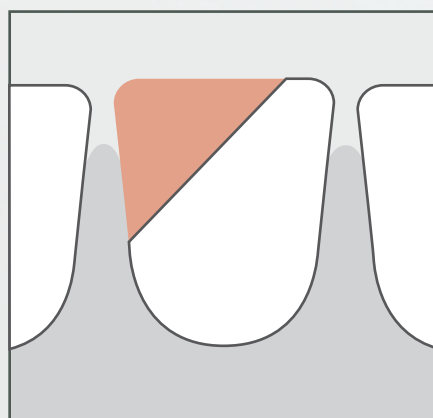

### Universal CLOUD Shade

Restore facial 1/2 of tooth thickness with universal CLOUD shade.

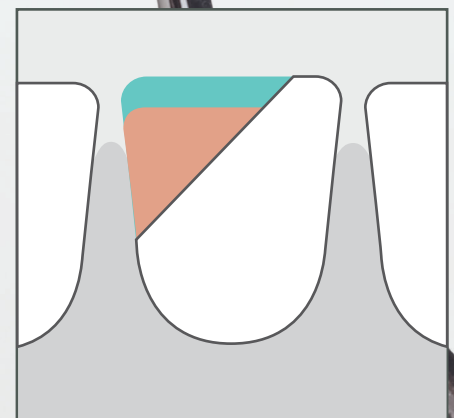

### Translucent Enamel Shade (Optional)

For a highly translucent incisal edge, optionally use the translucent enamel shade for the facial incisal edge increment.

1. VITA is not a registered trademark of Dentsply Sirona.

2. <http://www.melbourneorthodontics.com/educational/study-7-ways-smiles-affect-first-impressions/>

3. 2018 The Key Group, Inc.

# Neo Spectra™ ST Effects

## Universal Composite Restorative

Streamline your composite inventory with  
**Neo Spectra™ ST + Neo Spectra™ ST Effects Simplified Layering Technique.**



## Ordering Information

### Neo Spectra™ ST Effects

#### Universal Composite Restorative

#### High-Viscosity Refills

Shades

D1 (use with A1; A2)

D3 (use with A3; A3.5; A4)

E1 (use with A1; A2; A3; A3.5; A4)

#### Syringe

(1 x 3 g)

**607.01.952**

**607.01.953**

**607.01.951**



#### Neo Spectra ST Shade Guide

9 x Shade Tabs (A1; A2; A3; A3.5; A4; D1; D3; E1; BW)

**607.01.869**

For more information and to learn the technique, contact your Dentsply Sirona Restorative specialist, or visit [www.dentsplysirona.com/NeoSpectraSTEffects](http://www.dentsplysirona.com/NeoSpectraSTEffects)

<sup>1</sup> VITA is not a registered trademark of Dentsply Sirona.